www.elitebags.es

<https://youtu.be/CnBCLYCFatl>

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EN-1.DESCRIPTION AND INTENDED USE

Full body restraint straps used to immobilise patients on spine board, facilitating the safe transfer from point A to point B.

2. INSTRUCTIONS

Zone A: Upper body (blue strap)

Placed above the patient's chest then passed under the armpits. Threaded through the holes in the board situated at this height, continue wrapped under the board then up under the armpits again to the chest (DO NOT RESTRAIN ARMS). Close and adjust belt buckle over chest.

Zone B: Waist (orange, red & yellow straps)

Red strap (2) - Fitted first, placed over the waist and through the holes in the spine board at hip height and then looped under the board. The strap is then passed over the hip for a second time and inserted through the black ovals of the same strap. This insures the restraining force is correctly distributed. Close and adjust belt buckle over waist.

Yellow straps (3) - Fitted diagonally at the level of the iliac crest with each strap though the inside of the thigh at the anatomical hollow of the knee (popliteus muscle). Slide it zig-zagging up the thigh to the groin then pass it between the red strap and the patient, through the closest hole under the board. Tighten the strap inwards for buttock support in a perpendicular direction, towards the patient's head. Pull the straps out again through one or two holes higher up, depending on the patient's build. Close and adjust belt buckle over the iliac crest.

Orange strap (1) - Arm restraint. Placing the person's arms close to the wrists, the strap is passed over the ulna/radius. and secured. Close and adjust belt buckle.

Zone C: Legs & Feet (green strap)

The strap is wrapped around the feet in a figure of eight ready for immobilisation to the board.

1. Hold the toes, adjusting the straps on each foot one at a time.
2. Cross the straps in an X-shape over the tibia, pass through spinal board using the closest hole and loop back over, adding an additional cross below the knees.
3. Pass through holes in the spine board, then close and adjust belt buckle over the legs.

Auxiliary extension strap (black), used to lengthen either the blue, red or green straps by an additional 1m.

Make any final readjustments to the straps tightness needed for comfort and security.

3. SPECIFIC WARNINGS

Carefully check all buckles and straps BEFORE USE. If you detect loose threads, cuts, holes or tears in any strap, DO NOT USE OR REPAIR. Contact the manufacturer for replacements.

4. CLEANING AND MAINTENANCE

Wash dirty straps, including any that have been exposed to biological fluids, with warm water and neutral soap. Never use corrosive cleaning products. Always dry thoroughly before storage.